

TRINETRA GUIDE · CARDIAC SURGERY SUITE

DESKTOP INSTALLATION GUIDE

Clear installation steps for macOS Apple Silicon, macOS Intel and Windows x64.

VERSION

0.1.8

FOR

Research pilot users

UPDATED

15 September 2026

BEFORE YOU START

You need	Why
The correct 0.1.8 ZIP	Apple Silicon, Intel and Windows packages are different.
A licence code or .tri-license file	Studio, Casebook and Prep require activation.
Internet for first managed activation	The licence is bound to this device; normal offline use follows.
De-identified research data	Do not load identifiable clinical material into a pilot environment.

MACOS — APPLE SILICON OR INTEL

1

Choose the correct ZIP

On the Apple menu, select About This Mac. Apple M-series means Apple Silicon; an Intel processor means the Intel build.

2

Unzip

Double-click the ZIP. Move Trinetra Studio.app to Applications.

3

First launch

Control-click the app, choose Open, then choose Open again if macOS shows an unsigned developer warning.

4

Activate

Open Licence Settings, copy the installation ID if the owner needs it, then paste the licence code or import the .tri-license file.

5

Start in Casebook

Create or open a case, attach files, then choose Open in Studio or Open in Prep.

If macOS says the app is damaged

Delete that copy, unzip the original download again and use Control-click > Open. Do not run commands copied from an unknown website. Ask the Trinetra owner for a freshly verified package if the message remains.

WINDOWS X64

1

Download and extract

Right-click the ZIP, choose Extract All, and keep the extracted folder together.

2

Open the application

Run Trinetra Studio.exe from the extracted folder.

3

Handle SmartScreen

If Windows protects your PC, choose More info and Run anyway only when the file came from the Trinetra owner and its checksum matches.

4

Activate

Paste the supplied licence code or import the .tri-license file in Licence Settings.

5

Create the first case

Begin in Casebook, save the case locally, then open it in the required workspace.

LICENCE DISPLAY

What you see	Meaning
Days remaining	The number of whole or partial days before expiry.
Perpetual	No expiry date is encoded in the licence.
Device mismatch	The licence is already bound to a different device or was generated for another installation ID.
Disabled	The owner has remotely disabled this managed licence.
Offline allowance ended	Reconnect so Trinetra can refresh the managed licence status.

Check the download

Compare the package SHA-256 value with SHA256SUMS-0.1.8.txt before distribution or installation.